



Arginine Dihydrolase Broth

M619

Intended use

Arginine Dihydrolase Broth is used for detection of arginine dihydrolase producing microorganisms.

Composition**

Ingredients	Gms / Litre
Peptone	1.000
Sodium chloride	5.000
Dipotassium hydrogen phosphate	0.300
L-Arginine	10.000
Bromo cresol purple	0.016
Agar	3.000
Final pH (at 25°C)	6.0±0.2

**Formula adjusted, standardized to suit performance parameters

Directions

Suspend 19.31 grams in 1000 ml distilled water. Heat if necessary to dissolve the medium completely and distribute in 13x100 mm tubes. Sterilize by autoclaving at 115°C for 15 minutes. Allow the tubes to cool in an upright position.

Principle And Interpretation

Decarboxylase Media used for the detection of arginine dihydrolase and lysine and ornithine decarboxylase was first introduced by Moeller (1-3). Arginine Dihydrolase Broth is used for detection of arginine dihydrolase producing microorganisms. These types of media are used to differentiate bacteria on the basis of their decarboxylating activity towards the amino acids. Arginine decarboxylase enzyme is also known as Arginine dihydrolase. Arginine decarboxylase (or dihydrolase) production by various members of enteric bacteria aids in differentiating bacteria with closely related physiological characteristics (4). Bacteria producing arginine dihydrolase enzyme decarboxylates arginine present in this medium to putrescine. The production of amine, putrescine, elevates the pH. Bromo cresol purple is the pH indicator which forms purple colour in alkaline condition. Colour change from purple to yellow and then back to purple is positive reaction.

Peptone provide the necessary nutrients to the organisms while L-arginine stimulates the arginine dihydrolase synthesis.

Dipotassium phosphate buffers the medium while sodium chloride maintains the osmotic balance. In differentiation of *Enterobacteriaceae*, control tubes without arginine must be used. If the tubes give positive purple reaction the test is considered as negative.

Type of specimen

Clinical samples - Blood ; Food and dairy samples ; Water samples

Specimen Collection and Handling

For clinical samples follow appropriate techniques for handling specimens as per established guidelines (4,5).

For food and dairy samples, follow appropriate techniques for sample collection and processing as per guidelines (1,2,8).

For water samples, follow appropriate techniques for sample collection, processing as per guidelines and local standards.(3)

After use, contaminated materials must be sterilized by autoclaving before discarding.

Warning and Precautions :

In Vitro diagnostic Use only. Read the label before opening the container. Wear protective gloves/protective clothing/eye protection/ face protection. Follow good microbiological lab practices while handling specimens and culture. Standard precautions as per established guidelines should be followed while handling clinical specimens. Safety guidelines may be referred in individual safety data sheets

Limitations :

This medium is general purpose medium and may not support the growth of fastidious organisms.

Performance and Evaluation

Performance of the medium is expected when used as per the direction on the label within the expiry period when stored at recommended temperature.

Quality Control

Appearance

Light yellow to grey homogeneous free flowing powder

Gelling

Semisolid, comparable with 0.3% Agar gel.

Colour and Clarity of prepared medium

Purple coloured clear to slightly opalescent gel forms in tubes as butts

Reaction

Reaction of 1.93% w/v aqueous solution at 25°C. pH : 6.0±0.2

pH

5.80-6.20

Cultural Response

M619: Cultural characteristics observed after an incubation at 35-37°C for 18-24 hours.

Organism	Inoculum (CFU)	Growth	Motility	Arginine dihydrolase
Cultural Response				
<i>Enterobacter aerogenes</i> ATCC 13048 (00175*)	50-100	luxuriant	positive, growth away from stabline causing turbidity	negative reaction, yellow colour or no colour change
<i>Klebsiella pneumoniae</i> ATCC 13883 (00097*)	50-100	luxuriant	negative, growth along the stabline, surrounding medium remains clear	negative reaction, yellow colour or no colour change
<i>Proteus vulgaris</i> ATCC 13315	50-100	luxuriant	positive, growth away from stabline causing turbidity	negative reaction, yellow colour or no colour change
<i>Salmonella Typhi</i> ATCC 6539	50-100	luxuriant	positive, growth away from stabline causing turbidity	positive reaction, purple colour
<i>Salmonella Typhimurium</i> ATCC 14028 (00031*)	50-100	luxuriant	positive, growth away from stabline causing turbidity	positive reaction, purple colour
<i>Enterobacter sakazakii</i> ATCC 12868	50-100	luxuriant	positive, growth away from stabline causing turbidity	positive reaction, purple colour

Key : *Corresponding WDCM numbers.

Storage and Shelf Life

Store below 30°C in tightly closed container and the prepared medium at 2-8°C. Use before expiry date on the label.

Reference

1. Moeller, 1954, Acta Pathol. Microbiol. Scand., 34:102.
2. Moeller, 1954, Acta Pathol. Microbiol. Scand., 34:259.
3. Moeller, 1955, Acta Pathol. Microbiol. Scand., 36:158.
4. Gale and Epps, 1944, Biochem. J., 38:250.


Revision : 1 / 2011



In vitro diagnostic medical device



CE Marking

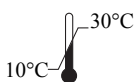

Storage temperature



Do not use if package is damaged



HiMedia Laboratories Pvt. Limited,
23 Vadhani Industrial Estate,
LBS Marg, Mumbai-86, MS, India



CE Partner 4U ,Esdoornlaan 13, 3951
DB Maarn The Netherlands,
www.cepartner4u.eu

Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.